

Guide to choosing Hemco Special Needs Harnesses, available in three sizes.

When choosing a size, consider that the harness is most effective when comfortably tight. All straps should be placed over bony areas, rather than soft tissue, the chest strap should sit across the centre of the breastbone, and the seat belt should sit over the wearer's hipbones.

There are three sets of loops at the back to adjust the position of the chest strap. There are two sets of loops at the front to adjust the position of the seat belt.


Note the Small Special Needs Harness must only be used for children weighing under 36kg, as the harness is designed to attach to the existing child seat anchorage in the vehicle.

The Medium and Large Special Needs Harnesses are designed to attach to an adult seat belt anchorage. The anchorage is made using an eye bolt for quick release. As there is usually no suitable existing anchor point in vehicles, it is recommended to have the anchorage kit installed by a competent mechanic at an appropriate Fitting station.

Shoulder strap

From top of shoulder
to lower waist loop.

Small: 46cm
 Medium: 51cm
 Large: 57cm

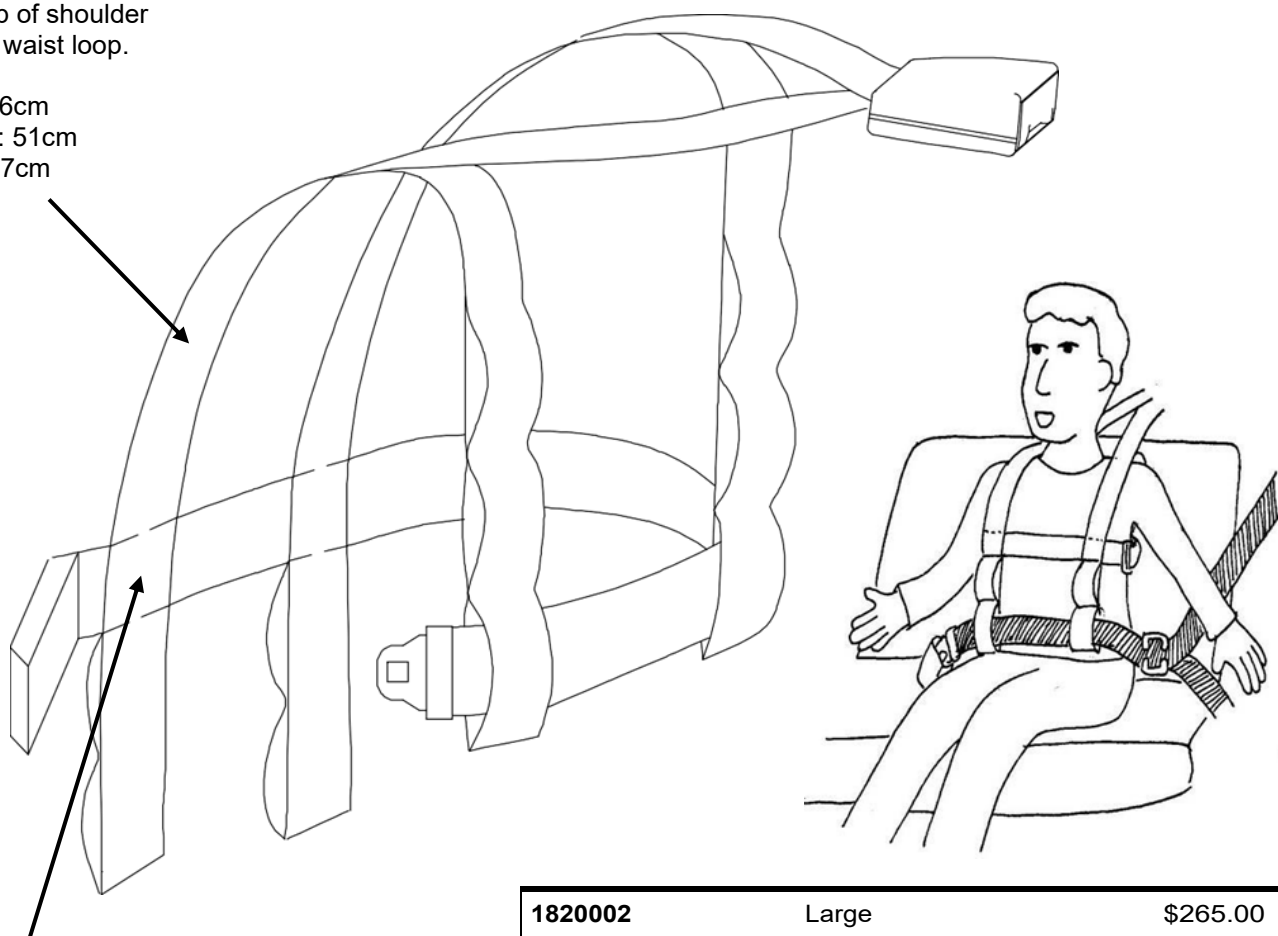

Chest Strap Length

Small: 90cm
 Medium: 110cm
 Large: 135cm

1820002	Large	\$265.00
1820001	Medium	\$265.00
1820000	Small	\$225.00
1820010-013	Anti-submarine strap	\$45.00